NOTE: Check all parts for shipping damage and test before use. In case of damage, DO NOT use. Contact your equipment supplier for further instruction.

SAVE THESE INSTRUCTIONS**SAFETY SUMMARY**

The following recommendations are made for the safe installation of the Infinity Seat:

GENERAL WARNINGS

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand the **Warnings, Cautions and Instructions**, contact a healthcare professional, dealer or technical personnel if applicable before attempting to install this equipment - otherwise, injury or damage may occur.

Skin condition should be checked very frequently after the installation of any new seat.

IMPORTANT INFORMATION

The best way to avoid problems related to pressure sores is to understand their causes and your role in a skin management program.

Your therapist and physician should be consulted if you have questions regarding individual limitations and needs.

All seats should be selected carefully. Working with your therapist, and physician is the best way to assure that a seat choice matches your individual needs.

As the needs of the individual become more complex, the seat evaluation becomes more important.

INTRODUCTION**INFINITY SEAT**

The Infinity seat (**seat**) is designed to provide pelvic stability, leg positioning and pressure management and is available in three (3) different base seats to suit the amount of positioning required by the user.

OUTER/INNER COVERS

The outer cover is made from a water repellant fabric to protect the foam. The top and sides are fabricated with a patented waterproof stretch fabric. The bottom is fabricated from a heavyweight, waterproof, non-skid fabric.

An inner water repellant cover is also provided for additional protection for the foam. Regular cleaning and inspection of the inner and outer covers is recommended. Refer to **CLEANING/INSPECTION** in this instruction sheet.

ATTACHING/REMOVING SEAT TO/FROM SEATING SURFACE (FIGURE 1)

NOTE: To remove the seat from a seating surface, reverse the following procedures.

NOTE: The seat attaches to the seating surface using the hook and loop fasteners on the bottom of the seat cover.

NOTE: The seating surface can be one (1) of the following: Seat upholstery, drop base or seat pan.

NOTE: The zipper on the seat should always face rearward.

WARNING

Make sure the hook and loop fasteners are securely attached to the seating surface and to the seat BEFORE using. If the seat is not secured to the seating surface, use caution when transferring in and out of the wheelchair.

1. Determine whether seating surface has the one (1) of the following and refer to one (1) of the following procedures:
 - A. Hook attachment strips.
 - B. Loop attachment strips.
 - C. No attachment strips.

HOOK ATTACHMENT STRIPS/NO ATTACHMENT STRIPS

1. Remove double sided hook from bottom of seat.
2. Align front edge of seat with front edge of seating surface.
3. Perform one (1) of the following:

HOOK ATTACHMENT STRIPS - Secure the seat to the seating surface, making sure that the strips of loop on the seat cover are securely attached to the strips of hook on the seating surface.

NO ATTACHMENT STRIPS - Place the seat on the seating surface.

LOOP ATTACHMENT STRIPS

1. Verify the double sided hook strips are securely attached to the loop attachment strips on the bottom of the seat.
2. Align front edge of seat with front edge of seating surface.
3. Secure the seat to the seating surface, making sure that the double sided hook is securely attached to the strips of loop on the seat cover and the seating surface.

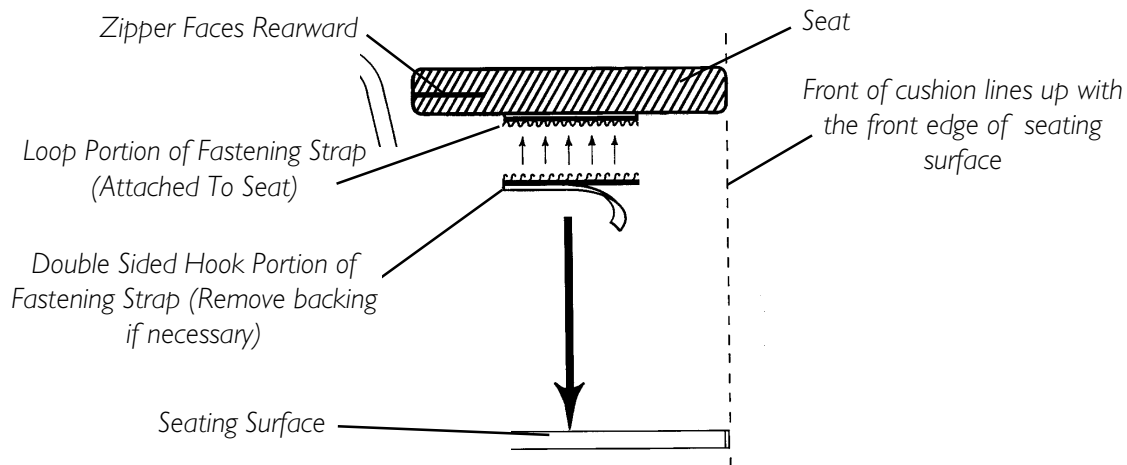
FIGURE 1 - ATTACHING/REMOVING SEAT TO/FROM SEATING SURFACE

REMOVING/INSTALLING SEAT COVERS

WARNING

The covers are designed to protect the foam from moisture and to provide fire retardancy. **DO NOT** use the seat without either the inner or outer cover. If the covers are torn, they must be replaced immediately.

REMOVING OUTER COVER (FIGURE 2)

1. Remove the seat from the seating surface. Refer to ATTACHING/REMOVING SEAT TO/FROM SEATING SURFACE in this instruction sheet.
2. Unzip the existing outer cover.
3. Remove seat (still inside inner cover) from existing outer cover.

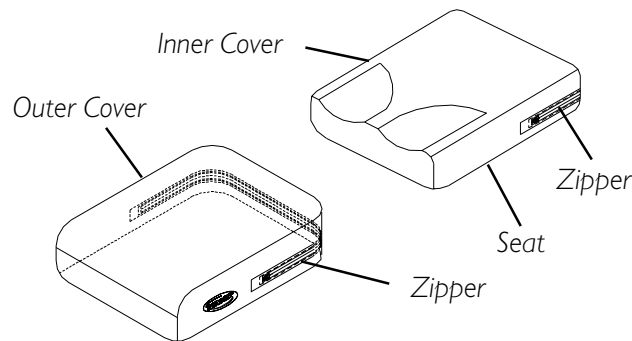
FIGURE 2 - REMOVING/INSTALLING SEAT COVERS - OUTER COVER

INSTALLING OUTER COVER (FIGURE 2)

1. Position the seat (inside inner cover) with the pelvic support layer towards the rear of the new/existing outer cover.

NOTE: The zipper is at the rear of the cover.

2. Slide seat (inside inner cover) into new/existing outer cover.
3. Zip the new/existing outer cover closed.
4. Reattach seat onto seating surface. Refer to ATTACHING/REMOVING SEAT TO/FROM SEATING SURFACE in this instruction sheet.

REMOVING INNER COVER (FIGURE 3)

1. Remove the seat from the outer cover. Refer to REMOVING/INSTALLING SEAT COVERS in this instruction sheet.
2. Unzip the existing inner cover.
3. Remove the seat from the existing inner cover.

INSTALLING INNER COVER (FIGURE 3)

1. Position the seat with the pelvic support layer towards the rear of the new/existing inner cover.

NOTE: The back surface of the seat is marked "REAR". The zipper is at the rear of the cover.

2. Slide the seat into the new/existing inner cover.
3. Zip the new/existing inner cover closed.

4. Reinstall the seat (inside inner cover) into the outer cover. Refer to REMOVING/INSTALLING SEAT COVERS in this instruction sheet.

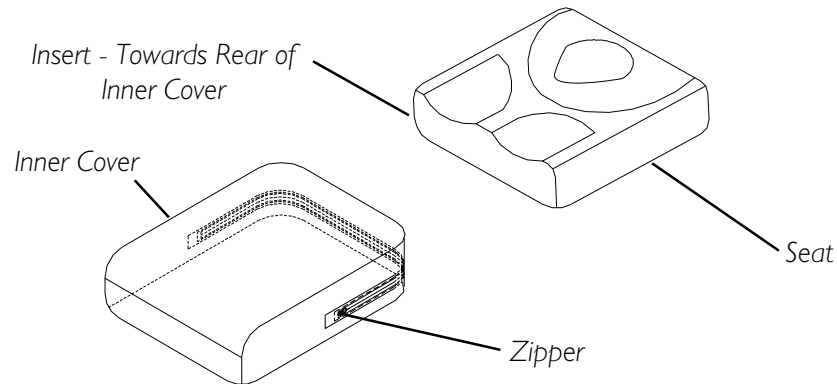
FIGURE 3 - REMOVING/INSTALLING SEAT COVERS - INNER COVER

OPTIONAL REMOVABLE LEG WEDGES (FIGURE 4)

NOTE: This procedure provides 1-inch of leg drop for foot propelling or accommodations of hip flexion limitations.

1. Remove the seat from the inner and outer covers. Refer to REMOVING/INSTALLING SEAT COVERS in this instruction sheet.
2. Remove the front leg wedge from the side of the seat that requires a lower seat to floor height. Save the leg wedge for future use.

NOTE: Leg wedges secure in place with hook and loop strips.

3. Reinstall the seat into the inner and outer covers. Refer to REMOVING/INSTALLING SEAT COVERS in this instruction sheet.

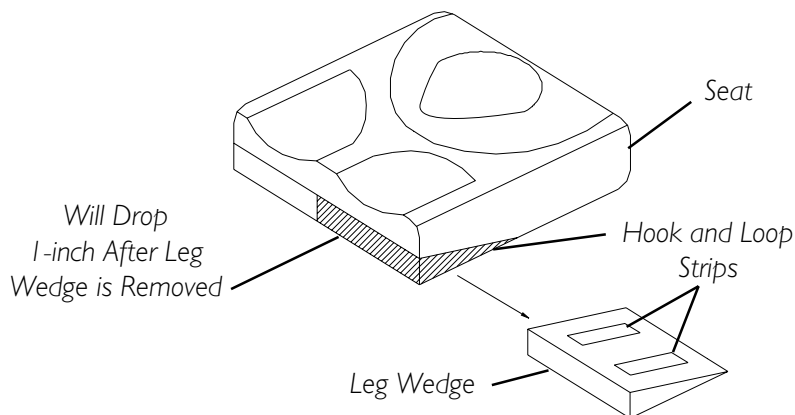
FIGURE 4 - OPTIONAL REMOVABLE LEG WEDGES

REMOVING/INSTALLING PELVIC SUPPORT LAYER (FIGURE 5)

NOTE: The seat comes with either a Viscoelastic Foam or a FloGel pelvic support layer.

REMOVING THE PELVIC SUPPORT LAYER

1. Remove seat from inner and outer covers. Refer to REMOVING/INSTALLING SEAT COVERS in this instruction sheet.
2. Lift UP on EXISTING pelvic support layer and remove from the seat.

INSTALLING THE PELVIC SUPPORT LAYER

1. Match loop strips on the bottom of the NEW/EXISTING pelvic support layer with the hook strips in the seat recess and press down.
2. If necessary, adjust the pelvic obliquity of the pelvic support layer. Refer to PELVIC OBLIQUITY ADJUSTMENT in this instruction sheet.
3. Reinstall seat into inner and outer covers. Refer to REMOVING/INSTALLING SEAT COVERS in this instruction sheet.

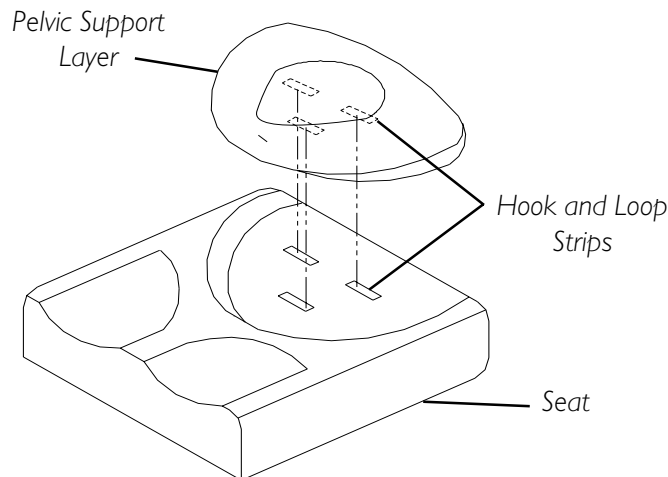
FIGURE 5 - REMOVING/INSTALLING INSERT

PELVIC OBLIQUITY ADJUSTMENT (FIGURE 6)

The pelvic obliquity adjustment can be made all at one time if the user can tolerate the positioning change or can be gently changed over time. This adjustment can be made in infinite increments, however, the maximum amount of pelvic obliquity is dependent on cushion width and contour.

1. Remove seat from inner and outer covers. Refer to REMOVING/INSTALLING SEAT COVERS in this instruction sheet.
2. Remove the pelvic support layer from the bottom seat layer.

NOTE: The pelvic support layer is secured to the bottom seat layer with hook and loop strips.


3. Move the pelvic support layer towards the side that needs to be raised to accommodate the obliquity.

NOTE: There is a centerline on the insert and five (5) indicator marks on the seat to use as guidelines: Neutral, Middle Right, Maximum Right, Middle Left and Maximum Left.

4. Make the following measurements:
 - A. From the top of the pelvic support layer on the high side to the seat (MEASUREMENT "A").
 - B. From the lower side of the pelvic support layer to the seat (MEASUREMENT "B").
5. Subtract MEASUREMENT "B" from MEASUREMENT "A" to calculate amount of obliquity achieved.
6. Adjust pelvic support layer until desired obliquity is obtained.

NOTE: Make sure hook and loop strips still secure together.

7. Press down on the insert to securely refasten the hook and loop strips.
8. Reinstall seat into inner and outer covers. Refer to REMOVING/INSTALLING SEAT COVERS in this instruction sheet.
9. Reattach the seat onto the seating surface of the wheelchair. Refer to ATTACHING/REMOVING SEAT TO/FROM SEATING SURFACE in this instruction sheet.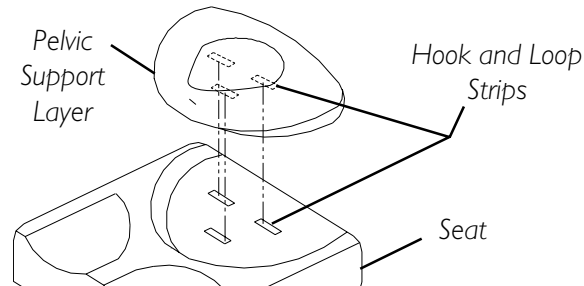

REAR VIEW

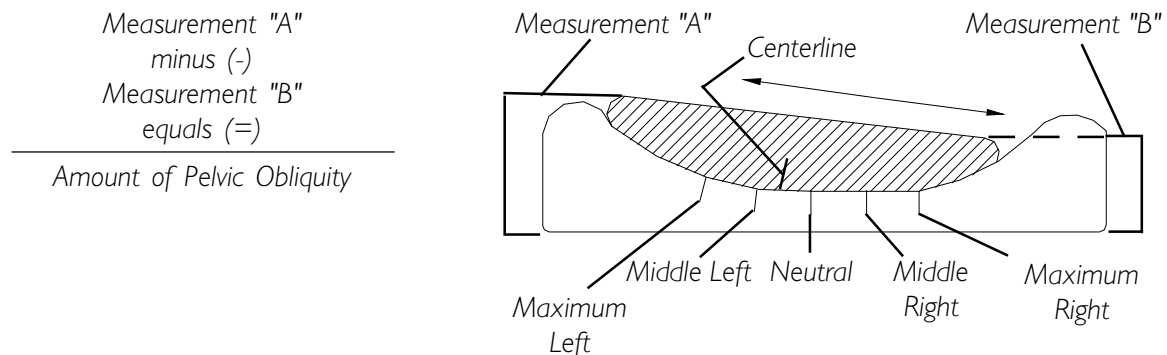
FIGURE 6 - PELVIC OBLIQUITY ADJUSTMENT

MODIFYING SEAT TO FIT ONTO DROP BASE (FIGURE 7)

NOTE: This procedure is required only if the seat does not sit flush on the drop base.

1. Remove seat from inner and outer covers. Refer to REMOVING/INSTALLING SEAT COVERS in this instruction sheet.
2. Without the inner or outer covers, place the seat on the drop base at the desired position.
3. Mark the points where the seat side wedges hit the drop base.

NOTE: Each point should be no more than 1-1/2-inches wide as shown in FIGURE 7.

4. Remove the seat from the drop base.
5. Notch the seat side wedges at the points determined in STEP 3.
6. Place the seat on the drop base at the desired position.

7. If necessary, repeat STEPS 4-6 until the seat sits flush on the drop base.
8. Reinstall seat into inner and outer covers. Refer to REMOVING/INSTALLING SEAT COVERS in this instruction sheet.
9. Reattach the seat onto the seating surface of the wheelchair. Refer to ATTACHING/REMOVING SEAT TO/FROM SEATING SURFACE in this instruction sheet.

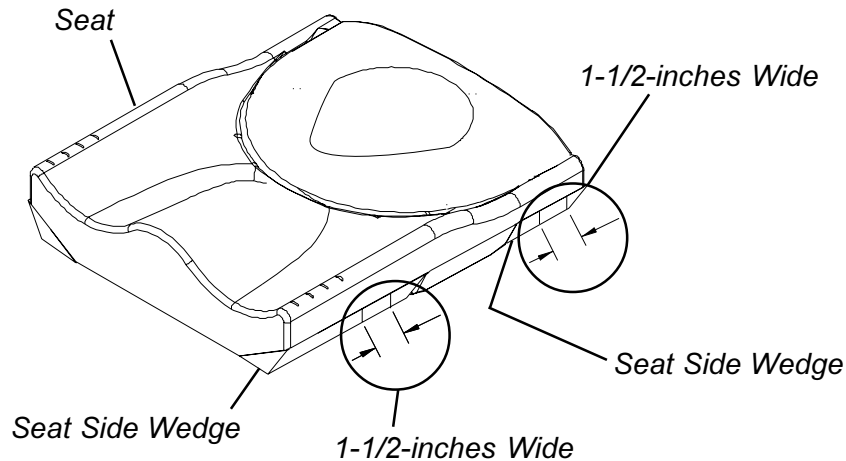
FIGURE 7 - MODIFYING SEAT TO FIT ONTO DROP BASE

SEAT INSPECTION AND CLEANING

INSPECTION

WARNING

DO NOT continue to use this product if any of the following problems as described are discovered. Corrective maintenance can be performed at or arranged through your equipment supplier.

1. **Once a week**, visually inspect all parts, including hardware, upholstery materials, foams (if accessible), and plastics for deformation, corrosion, breakage, wear and/or compression.

CLEANING

1. Perform one (1) of the following **ONCE A WEEK**:

Inner Cover.

Hand clean the water repellent inner cover using a soft cloth, warm water and a mild soap solution. Rinse off and hang dry.

Outer Cover.

CAUTION

MACHINE WASH seat cover in cold water on gentle cycle using a mild detergent. **DO NOT** use fabric softeners or bleach. **DO NOT** machine dry. **Air dry ONLY. DO NOT** dry in sun.

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for two (2) years on seat and 90 days on cover of use by original purchaser. With regard to the original purchaser/user only, Invacare warrants its metal fabricated hardware components to be free from defects in materials and workmanship for the lifetime of the product except any wood, foam plastic or upholstered components. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN, THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

UNITED KINGDOM LIMITED WARRANTY

This warranty is extended only to the original purchaser of our products and does not affect statutory rights.

Invacare UK / Invacare Service Representative warrants the foam cushion to be free from defects for a period of two (2) years from the date of purchase. The cushion cover is warranted for a period of ninety (90) days from date of purchase. With regard to the original purchaser/user only, Invacare UK /Invacare Service Representative warrants its metal fabricated hardware components to be free from defects in materials and workmanship for the lifetime of the product except any wood, foam plastic or upholstered components. If within such warranty period the product shall be proven to Invacare UK's / Invacare Service Representative reasonable satisfaction to be defective, it shall be repaired or replaced, at Invacare UK's / Invacare Service Representative option ; Invacare UK's / Invacare Service Representative sole obligation, and your exclusive remedy under this warranty being limited to such repair or replacement.

For warranty service, please contact Invacare UK's Service Department / Invacare Service Representative during normal working hours. Upon receiving notice of an alleged defect in a product, Invacare UK / Invacare Service Representative will issue a serialized return authorization. The product shall then be returned to either Invacare plant or service center as specified by Invacare UK / Invacare Service Representative. Defective component part(s) must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without prior consent.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY DOES NOT COVER NORMAL WEAR AND TEAR AND SHALL NOT APPLY TO PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE UK / INVACARE SERVICE REPRESENTATIVE.

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© 2001 Invacare Corporation
Invacare is a registered trademark of
Yes, you can. is a trademark of Invacare
Invacare Corporation www.invacare.com